



Micromedex[®] Tips and Tricks for Physicians

Micromedex Assistant and Keyword Search


Use the Micromedex Assistant to get quick answers to drug information without having to navigate to find the section in the monograph. The Micromedex Assistant has been trained to answer with **Quick Answers** content, as well as **IV compatibility** and **drug interactions**. Quick links at the end of the answer take you to Quick or In-Depth Answers within the monograph.

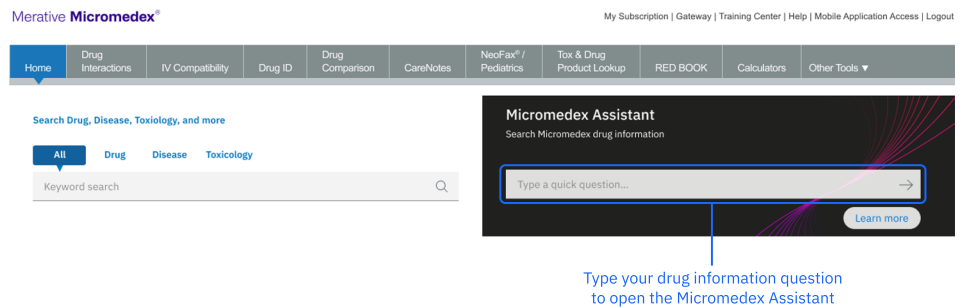


1. Ask a question

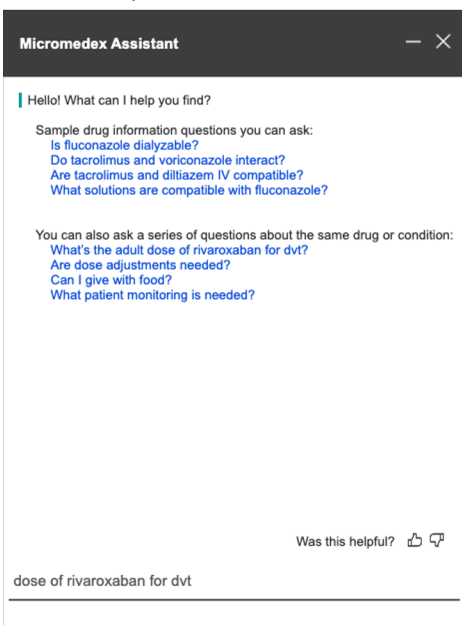

2. Get the answer. Ask another

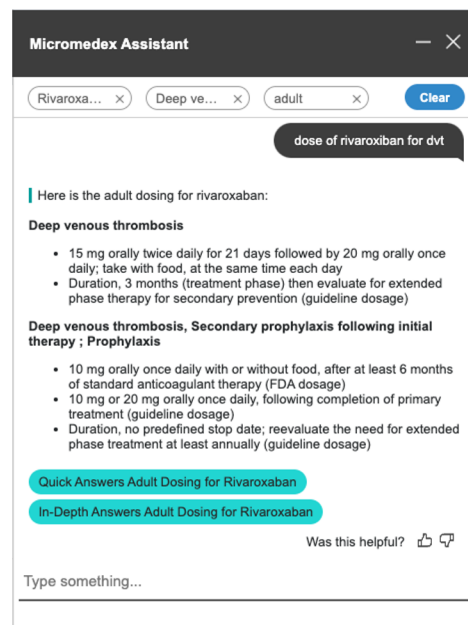

3. Get the answer

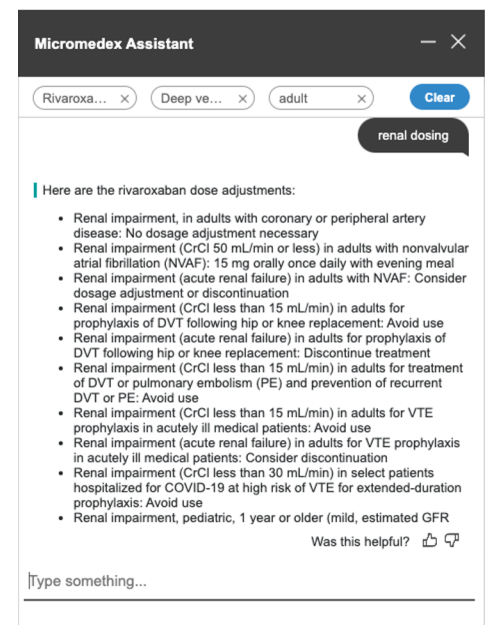

More sample questions the Micromedex Assistant can answer:

can i crush ditropan	interaction coumadin and aspirin	drugs that treat [condition]	side effects of caffiene	pediatric dose of augmentin for otitis
trade name for [drug]	drug class of bactroban	is [drug] safe in pregnancy	pediatric fda approved uses for [drug]	is [drug] compatible with d5w
can i push ancef	monitoring for lasix	adrs for [drug]	bbw [drug]	monitoring for [drug]

Micromedex with Keyword Search

Use the Keyword Search when you have general questions or want to read about a drug or condition.

Merative **Micromedex**[®] My Subscription | Gateway | Training Center | Help | Mobile Application Access | Logout

Home | Drug Interactions | IV Compatibility | Drug ID | Drug Comparison | CareNotes | NeoFax[®]/Pediatrics | Tox & Drug Product Lookup | RED BOOK | Calculators | Other Tools ▾

Search Drug, Disease, Toxicology, and more

All | Drug | Disease | Toxicology

Keyword search

Micromedex Assistant
Search Micromedex drug information

Type a quick question...

Learn more

Use the Keyword search for general, disease, or toxicology questions

Search Term/Phrase	Description
[YYYY]	Search by specific year (1995->). Returns the Drug Consult on drugs approved in that year. To see all: search on 'new drug approvals.'
Acronyms	Returns the Drug Consult, <i>Chemotherapy Acronyms and Dosing</i> with Alphabetical list of Chemotherapy acronyms and dosing regimens. Can also search specific regimens
ABVD	Returns drug consult, <i>ABVD – Used for Hodgkin’s Disease Chemotherapy regimen.</i>
Clinical approach to	Returns links to toxin-induced disease states (e.g., Hyperthermia, Hypotension).
Chemotherapy	Returns Drug Consults such as <i>Chemotherapy Dosing in Obese Patients, Chemotherapy and radiotherapy Treatment Guidelines for Nausea and Vomiting</i> and others.
Drugs that treat aspergillosis	Returns drugs that treat aspergillosis based on Efficacy Rating. Try 'drugs that treat' for other indications.
Drugs that cause Hyperkalemia	Returns a list of drugs associated with hyperkalemia. Click on 'T' & choose tacrolimus; see tacrolimus adverse effects. Under Serious, Endocrine: view incidence for hyperkalemia. For more detailed evidence on this ADR, click In-Depth Answers, Endocrine/Metabolic, Hyperkalemia.
Guidelines	Returns an extensive list of Drug Consults summarizing treatment guidelines covering infectious diseases, cardiovascular, neurologic, immunologic, endocrine/metabolic, and more.
Drug interactions* (include * as wild card)	Returns list of individual Drug Consults reviewing specific inhibitors, inducers, and substrates.
Opioid	Returns list of drugs in the opioid class.
Opioid* (include * as wild card)	Filter All Results by Drug. Returns Drug Consults focused on treatment of pain, opioid-induced constipation, opioid withdrawal, and others.
Extravasation	Returns results that include Drug Consults, <i>Cytotoxic Drug Extravasation Therapy and Non-Cytotoxic Drug Extravasation Therapy.</i>
Beers	Returns Drug Consult, <i>Beers Criteria–A Summary of Potentially Inappropriate Medication Use Among the Elderly.</i> Try searching stopp to access START/STOP Drug Consult.
AHA	Filter All Results by Drug. Returns Drug Consults covering a variety of guidelines from the AHA. Try 'CDC', 'IDSA', 'ACCP', 'ACR'.
Mobile apps	Passwords are found under the Mobile Application Access section on the home page. Passwords for the NeoFax[®] and Pediatrics apps are found in the NeoFax application, under Mobile at the top right of the display.

Get connected

24/7 Phone Support in the United States & Canada: [1-877-843-6796](tel:1-877-843-6796)

Outside the United States and Canada: [1-303-354-4100](tel:1-303-354-4100)

Email Micromedex at ProductSupport@Merative.com

© Copyright Merative US LP. 2023
Merative
100 Phoenix Drive
Ann Arbor, Michigan 48108

Produced in the United States of America
December 2023

Merative, the Merative logo and Micromedex are trademarks of Merative in the United States, other countries or both. All other company or product names are registered trademarks or trademarks of their respective companies.

This document is current as of the initial date of publication and may be changed by Merative at any time. Not all offerings are available in every country in which Merative operates.

The information in this document is provided "as is" without any warranty, express or implied, including without any warranties of merchantability, fitness for a particular purpose and any warranty or condition of non-infringement. Merative products are warranted according to the terms and conditions of the agreements under which they are provided.

The client is responsible for ensuring compliance with all applicable laws and regulations applicable to it. Merative does not provide legal advice or represent or warrant that its services or products will ensure that the client is in compliance with any law or regulation. The performance data and client examples cited are presented for illustrative purposes only. Actual performance results may vary depending on the specific configurations and operating conditions. It is the user's responsibility to evaluate and verify the operation of any other products or programs with Merative product and programs.

Statement of Good Security Practices: IT system security involves protecting systems and information through prevention, detection and response to improper access from within and outside your enterprise. Improper access can result in information being altered, destroyed, misappropriated or misused or can result in damage to or misuse of your systems, including for use in attacks on others. No IT system or product should be considered completely secure and no single product, service or security measure can be completely effective in preventing improper use or access. Merative systems, products and services are designed to be part of a lawful, comprehensive security approach, which will necessarily involve additional operational procedures, and may require other systems, products or services to be most effective. Merative does not warrant that any systems, product or services are immune from, or will make your enterprise immune from, the malicious or illegal conduct of any party.